



Dosimetry and Quality Assurance Tools for Particle Therapy

Radu A. Vasilache, Ph.D.

Canberra Packard Central Europe GmbH

Abstract

In the latest years, particle therapy – i.e. proton and heavy ion therapy (especially carbon) is gaining recognition as a valuable therapy method due to the possibility to deposit the beam energy within a small determined target volume, while the energy imparted before the location of the Bragg peak has a low percentage compared to the total dose.

In order to be usable as a valid therapeutic tool, appropriate dosimetry and QA tools need to be developed. In comparison to the tools developed for photon and electron beams, a number of challenges have to be met, such as:

- High mechanical precision and small scanning increments are required, especially for depth dose measurements (e.g. 10 μm

increments)

- High instantaneous dose rates are encountered, however, the radiation can be considered as *continuous* radiation

The paper presents a number of such dosimetry and QA tools developed by PTW, that allow an accurate in-beam dosimetry, beam analysis, finding of the Bragg peak, and it shows some results in determining the Bragg peak in proton and carbon ion beams.

Canberra Packard
CENTRAL EUROPE GMBH

Dosimetry & Quality Assurance Tools for Particle Therapy

Dr. Radu Vasilache, Canberra Packard CE GmbH
Dr. Edmund Schüle, PTW Freiburg GmbH

1


> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009

CANBERRA

Canberra Packard
CENTRAL EUROPE GMBH

Particle Accelerators

Classical Cyclotron

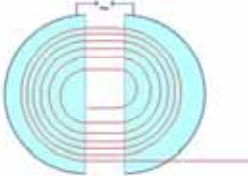

$$qvB = \frac{m_0 v^2}{r}$$

Lorentz force =
centrifugal force

$$\omega = \frac{q}{m_0} B$$

$$E_{kin} = \frac{r^2 q^2 B^2}{2m_0} = \frac{m_0}{2} \omega^2 r^2$$

- ▶ Frequency ω independent on radius, can be kept constant
- ▶ Energy proportional to squares of radius and magnetic field
 - spiral gets narrower with increasing radius/energy
 - higher energies require greater r and B and therefore higher costs
- ▶ As r_{max} and ω are fixed for a specific cyclotron, only one energy for a given type of particle is possible
 - degrader required

2


> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009

CANBERRA

Canberra Packard
CENTRAL EUROPE GMBH

Particle Accelerators

Relativistic Cyclotron

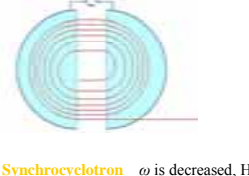


$$\omega = \frac{q}{m} B$$

▶ As m increases with energy,
either ω must be decreased,
or B must be increased accordingly

- ▶ **Synchrocyclotron** ω is decreased, HF is synchronized with orbit time of particle
 - pulsed operation required
 - Example: Still River Systems
- ▶ **Isochroncyclotron** B is increased with increasing r to keep $B/m = \text{const}$
 - continuous operation
 - requires spiral sectors with different B to focus the beam
(Thomas Cyclotron, AVF - azimuthally varying field)

Examples: Varian/Accel 250 MeV p^+
IBA 230 MeV p^+



3


> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009

CANBERRA

Canberra Packard
CENTRAL EUROPE GMSH

Particle Accelerators

Synchrotron

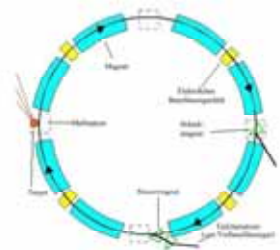

$$\omega = \frac{v}{r} = \frac{q}{m} B$$

- Keep r constant to save iron in the magnets, increase ω and B by time

- Frequency and magnetic field are synchronized with particle speed
 - pulsed operation required
 - as $B(t)$ varies slowly spills are available only every 1 – 10 seconds
 - variable energy

Examples: Siemens 220 MeV p^+ or 430 MeV/u ^{12}C
CERN 28 GeV p^+ at $r = 70$ m, 7 TeV p^+ at $r = 4.3$ km

4

> Radiotherapy with Neutron and Proton Beams, Predreal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GMSH

Dosimetry measurements: What is so special ?

- High mechanical precision and small scanning increments are required, especially for depth dose measurements (e.g. 10 μ m increments)
- High instantaneous dose rates are encountered, however, the radiation can be considered as *continuous* radiation

5


> Radiotherapy with Neutron and Proton Beams, Predreal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GMSH

Absolute Dosimetry

UNIDOS^{weblne} and Solid State Phantoms

- UNIDOS^{weblne} dosimeter with TCP/IP interface
- PMMA slab phantom type 40041 (25 x 25 x 27) cm³, with temperature sensor embedded in Farmer chamber plate
- RW3 slab phantom 29672 (30 x 30 x 30) cm³; 45° holding device
- Readout unit L981179 for phantom temperature and air pressure, with TCP/IP interface



6


> Radiotherapy with Neutron and Proton Beams, Predreal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GMSH

Absolute Dosimetry

Ionization Chambers and Check Sources

- Farmer chambers 30010, 30013, 30011
- Advanced Markus chamber 30045
- Roos chamber 34001
- Radioactive check sources 48012, 48010
- Water phantom 41023

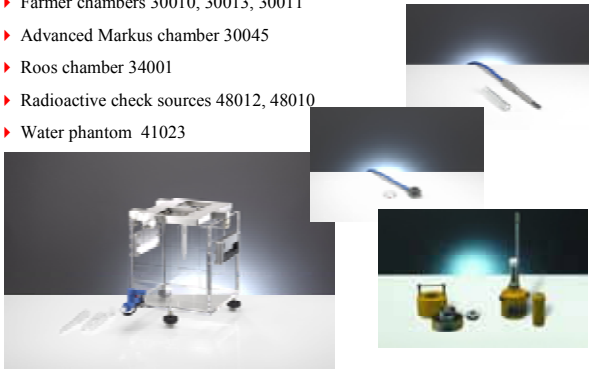

7



> Radiotherapy with Neutron and Proton Beams, Predreal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GmbH

Water Phantom

MP3-P Water Phantom and Reservoir

- Enhanced mechanical precision
- Measuring range (350 x 250 x 380) mm
- Entrance window (250 x 250 x 5) mm
- MEPHYSTO mc^2 software

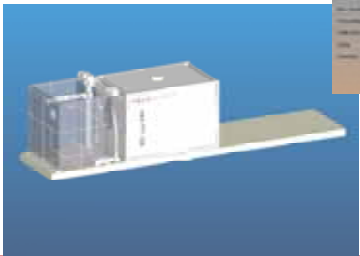Designed for Siemens PT

8


> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GmbH

Water Phantom

MP3-P Electronics and Chambers

- TBA Control Unit with Control Pendant
- TANDEM Dual Channel Electrometer
- Bragg Peak Chambers 34073 (Roos Type), 34070 (Standard) and 34080 (Thin Window)



Chambers 34073 used by DKFZ/GSI in Heavy Ion beams

9

> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GmbH

Water Phantom

Bragg Peak Chambers

- Types 34070 and 34080 for MP3-P measurements
- Type 34073 for MP3-P measurements in Heavy Ion beams
- Types 34080 and 34082 used in PEAKFINDER

	Standard T34070	Roos-Type T34073	Thin Window T34080	Thin Window Monitor T34082
Volume	10.5 cm³	2.5 cm³	10.5 cm³	5.2 cm³
Outer diameter/height	104 mm/13 mm	68 mm/10.4 mm	104 mm/10 mm	80 mm/10 mm
Entrance window thickness	3.47 mm	1.13 mm	0.62 mm	0.62 mm
Exit window thickness	7.48 mm	1.46 mm	1.16 mm + 6 mm air	1.16 mm + 6 mm air
Electrode diameter	81.6 mm	39.6 mm	81.6 mm	57.6 mm
Electrode spacing	2 mm	2.013 mm	2 mm	2 mm
Chamber voltage	400 V	400 V	400 V	400 V
Waterproof	yes	yes	no	no
For use at different depths	yes	no	no	no

Chambers 34073 used by DKFZ/GSI in Heavy Ion beams

10

> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GmbH

Water Phantom

MP3-P Dual Chamber Holders


- L981422 for two Bragg Peak chambers 34073 *)
- L981423 for Bragg Peak chambers 34070 (field) and 34080 (reference) *)




*) For use in horizontal beams only

11


> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009





Water Phantom

Bragg Peak Measurement Example

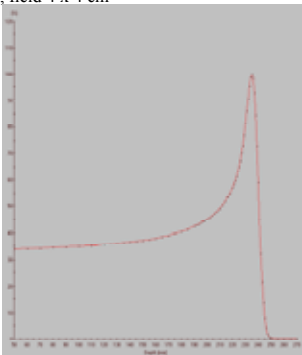- ▶ Bragg Peak of 225 MeV proton beam, field 4 x 4 cm²
- ▶ MP3 system, Advanced Markus chamber, horizontal beam
- ▶ 1 mm increments
- ▶ Measured at MD Anderson, USA, March 2006





12
> Radiotherapy with Neutron and Proton Beams, Predreal, February 27th – March 1st 2009





Water Phantom

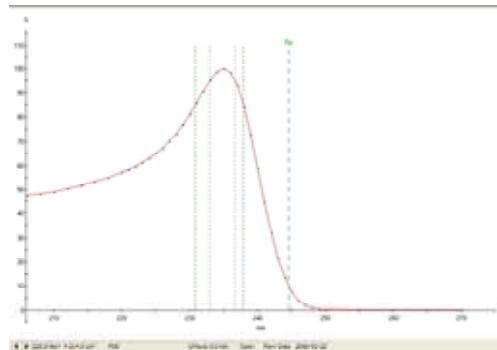Bragg Peak Evaluation





13
> Radiotherapy with Neutron and Proton Beams, Predreal, February 27th – March 1st 2009





Water Column

PEAKFINDER

- ▶ Can be used in any spatial orientation, closed construction
- ▶ Measuring depths (20 ... 360) mm, diameter 100 mm
- ▶ Spatial resolution 10 μm, accuracy 100 μm, ≤ 10 mm/s
- ▶ Servo control unit with feed-back system for positioning check
- ▶ TANDEM Dual channel electrometer, Gate input for spill-by- spill measurements
- ▶ PeakScan software

14
> Radiotherapy with Neutron and Proton Beams, Predreal, February 27th – March 1st 2009




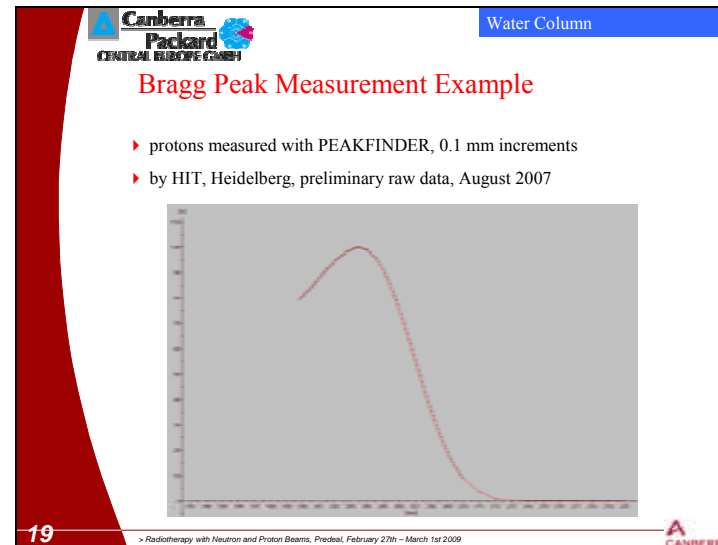
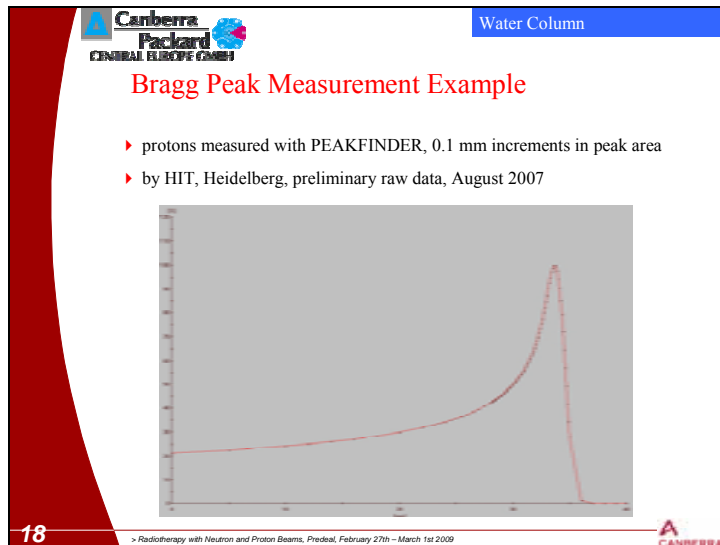
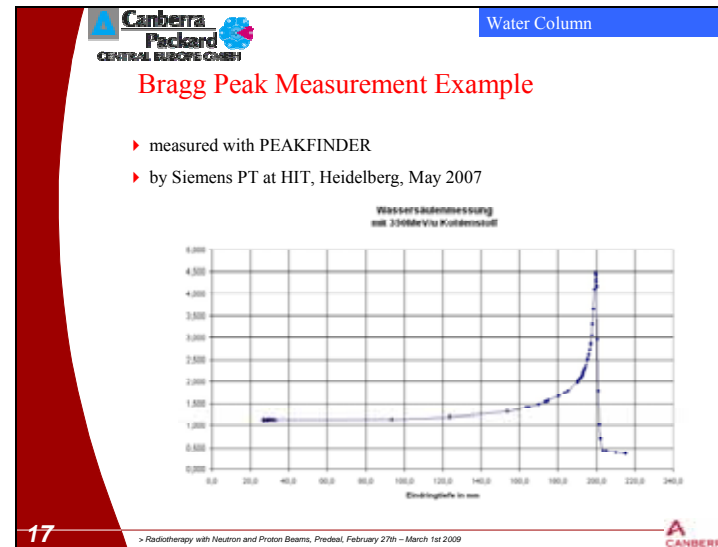
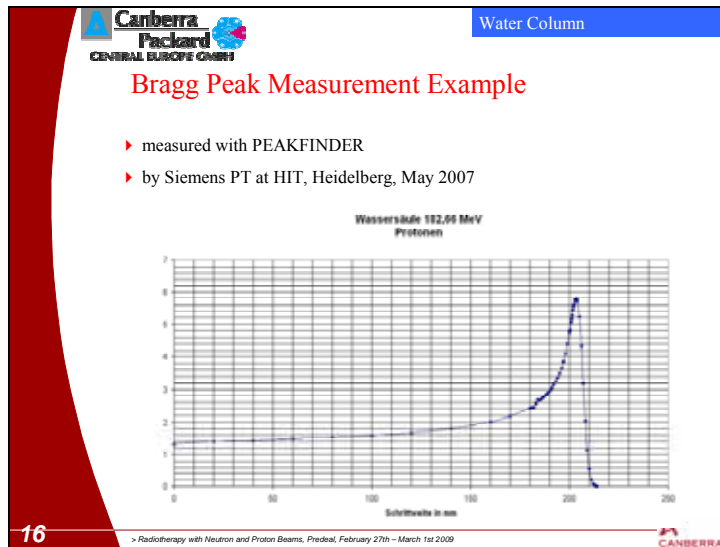
Water Column

PEAKFINDER w/o Cover

- ▶ Built-in thin window Bragg Peak chambers T34082 (Monitor) and T34080 (Field)
- ▶ Column windows 3 mm Quartz glass



15
> Radiotherapy with Neutron and Proton Beams, Predreal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GmbH

Film Dosimetry

Film Scanner and Holder

- Digital film scanner EPSON 10000XL, A3 format
- FilmAnalysis software for MEPHYSTO *mc*²
- Holder 40040 for X-Omat V films (265 x 325) mm²; optional 45° holding device



Holder designed for Siemens PT

20


> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GmbH

1D Dose Distribution

MULTIDOS with 12 PinPoint Chambers

- MULTIDOS 12-channel electrometer, with detector connection box
- Linear Detector Holder, mounts to MP3-P system, holds up to 24 PinPoint chambers (0.03 cm³, T31015), covers 250 mm measuring length

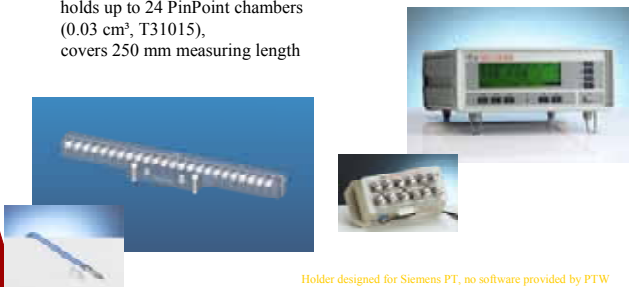

Holder designed for Siemens PT, no software provided by PTW

21


> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GmbH

2D Dose Distribution

2D-ARRAY *xdr*

- 729 vented ionization chambers, 27 x 27 field size
- dynamic range adapted to high dose rates (1 ... 100 Gy/min)
- extended high voltage (900 V) for better saturation
- VeriSoft software

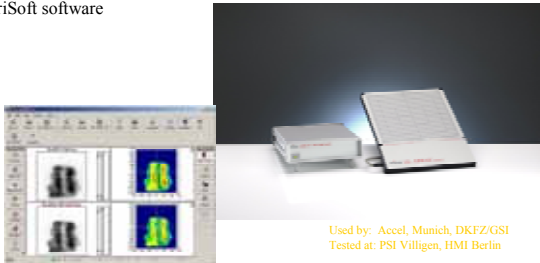

Used by: Accel, Munich, DKFZ/GSI
Tested at: PSI Villigen, HMI Berlin

22


> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009

Canberra Packard
CENTRAL EUROPE GmbH

3D Dose Distribution

MULTIDOS with 3D Detector Block

- MULTIDOS 12-channel electrometer, with detector connection box
- 3D Detector Block, mounts to MP3-P system
- Holds up to 24 PinPoint chambers (0.03 cm³, T31015)
- Detector spacing 10 mm and 12 mm




Used at GSI (Jäkel, Hartmann et al), no software provided by PTW

23

> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009

Positioning Phantom

- ▶ PMMA phantom type 43029 (150 mm diameter, 150 mm height), with internal markers
- ▶ Checks positions in digital images (DRR, CT, MR) versus laser coordinates

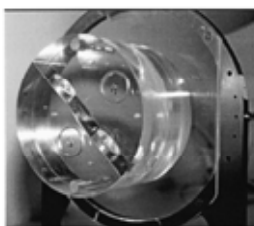

Used at GSI (Jäkel, Hartmann et al)

24

> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009



Sphere with Micrometer Drives

- ▶ 2 mm sphere, adjustable in 1 μm increments; optional 5 mm sphere with adjustment cube
- ▶ Checks isocentric rotation of treatment couch
- ▶ Checks isocentric beam position at different energies
- ▶ Radiation shadow is monitored by film



Used at GSI (Jäkel, Hartmann et al)



25

> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009



Robot Calibration Plate

- ▶ Accuracy of engravings 0.1 mm
- ▶ Checks congruence between laser positions and robot coordinate system
- ▶ Checks linear and angular scales

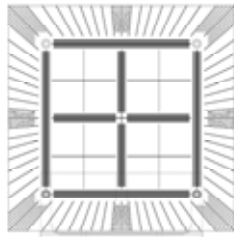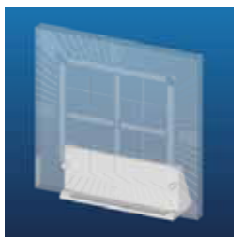

Designed for Siemens


26

> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009



Inhomogeneity Phantom

- ▶ PMMA phantom, 200 mm in diameter
- ▶ 11 inhomogeneities, diameter 20 mm, length 10 mm. Densities from 0.001 ... 1.82 g/cm³
- ▶ Adaption plate for up to 24 PinPoint chambers
- ▶ Can be mounted to entrance window of MP3-P water tank for measurements with the 3D detector block
- ▶ Checks the system's ability to irradiate inhomogeneous targets

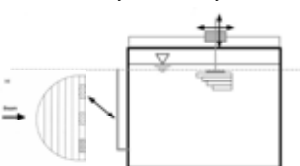

Used at GSI (Jäkel, Hartmann et al)

27

> Radiotherapy with Neutron and Proton Beams, Predeal, February 27th – March 1st 2009



Thank you!

